
ColsigenreFresh™

Directions for use / Instrucciones de uso / Mode d'emploi

VEGAN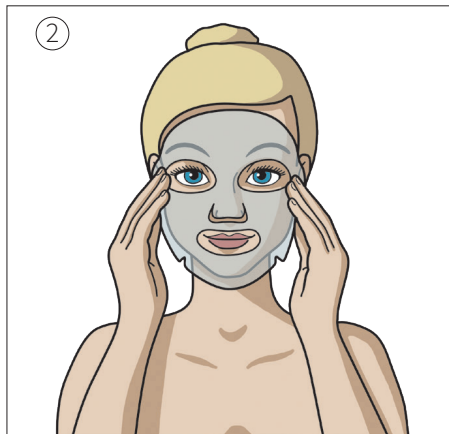

FRAGRANCE-FREE

5 single-use masks,
0.84 fl oz (25mL) each

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Scan me!

us.colsigen.com

 **Stratpharma** Switzerland
Heal Better™

CF000US00PI01-0725

[EN] Consumer information

Colsigen Refresh is a restorative face mask formulated with Humanized Type III Collagen.

Colsigen Refresh soothes, deeply hydrates, and rejuvenates sensitive or compromised skin, repairing the damage from cosmetic procedures and daily environmental stresses.

How Colsigen Refresh works?

This mask, designed with advanced fabric technology to conform to facial contours, helping to create a barrier that moisturizes evenly and ensures comfort during use.

Colsigen Refresh supports the skin's natural moisture barrier and repair processes by delivering hydration to areas of skin that are dry, irritated, or visibly compromised. It promotes a soothing environment that helps reduce redness and discomfort post-procedure.

Colsigen Refresh is gentle enough for everyday use.

Intended purpose (intended use)

Colsigen Refresh is a transient protective barrier, which improves hydration and supports skin repair.

Indications

Intended to be used daily or as needed on all types of skin. Indicated for dry, irritated and compromised skin. Indicated for use following cosmetic procedures such as laser resurfacing, microdermabrasion, microneedling and superficial peels.

Intended users

Colsigen Refresh is indicated for adults, children, and people with sensitive skin.

Contra-indications

- Not for use by individuals allergic to any of the ingredients.
- Not suitable for highly exuding or infected wounds.

Warnings and precautions

- For external use only.
- Avoid contact with the eyes.
- If irritation occurs, discontinue use and consult your physician.
- Do not use if the package is damaged.
- Do not use after the expiration (EXP) date printed on the sachet.

Ingredients: Mask: 100% plant fiber. Gel:

Purified water, Glycerin, Recombinant type III Humanized Collagen, Carbomer, Methylparaben, Disodium EDTA, Triethanolamine

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Stratpharma AG, Aeschenvorstadt 57, CH-4051 Basel, Switzerland / Suiza / Suisse