
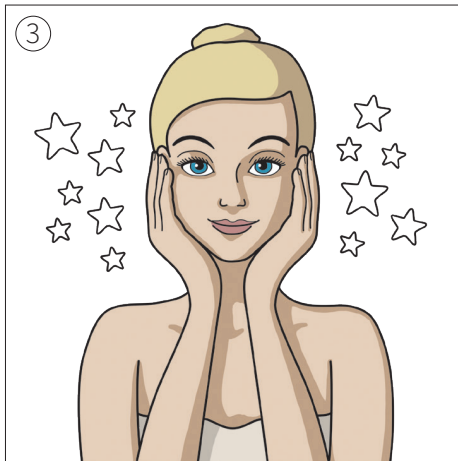
Directions for use / Instrucciones de uso / Mode d'emploi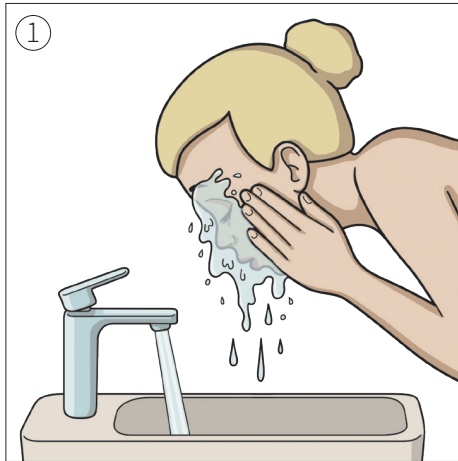

us.colsigen.com

MD

Medical device
Dispositivo médico
Dispositif médical

Consult instructions for use
Consultar instrucciones de uso
Consulter le mode d'emploi

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3.4 fl oz
(100 mL)

CM000US00PI01-0725

Stratpharma Switzerland
Heal Better™

[EN] Consumer information

Colsigen Mist is a restorative mist formulated with Humanized Type III Collagen.

Colsigen Mist soothes, deeply hydrates, and rejuvenates sensitive or compromised skin, repairing the damage from cosmetic procedures and daily environmental stresses.

How Colsigen Mist works?

Colsigen Mist assists hydration to facial contours by creating a barrier that helps to distribute moisture evenly while ensuring comfort during use.

This mist is powered by nitrogen technology, making it plane safe and free from butane or other flammable gases.

Colsigen Mist supports the skin's natural moisture barrier and repair processes by delivering hydration to areas of skin that are dry, irritated, or visibly compromised. It promotes a soothing environment that helps reduce redness and discomfort post-procedure.

Colsigen Mist is gentle enough to be used throughout the day, as needed.

Intended purpose (intended use)

Colsigen Mist provides a transient barrier that hydrates and supports the skins repair.

Indications

Indicated to be used daily or as needed on all types of skin.

Indicated for dry, irritated and compromised skin.

Indicated for use following cosmetic procedures such as laser resurfacing, microdermabrasion, microneedling and superficial peels.

Intended users

Colsigen Mist is indicated for adults, children, and people with sensitive skin.

Contra-indications

- Not for use by individuals allergic to any of the ingredients.
- Not suitable for highly exudating or infected wounds.

Warnings and precautions

- External use only.
- Avoid contact with the eyes.
- If irritation occurs, discontinue use and consult your physician.
- Do not use if the package is damaged.
- Do not use after the expiration (EXP) date printed on the can.

Ingredients: Purified Water, Glycerin, Sodium dihydrogen phosphate, Recombinant type III Humanized Collagen, Methylparaben, Sodium chloride, Phenoxyethanol

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